


H. Pylori SA

This is not a complete instruction for use. For more detailed instructions, please refer to IFU.

Test Components

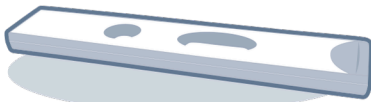

Test cartridge



Detector tube (Granule)



Extraction buffer tube



ID chip




Ensure that the lot number of the components match.

LOT Matches **LOT**




If extraction buffer tube are stored in refrigerator, keep the sample collection tube at room temperature for 30 minutes just prior to the test.

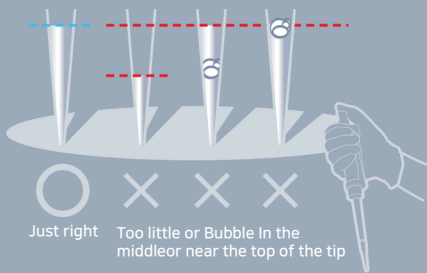


I. Specimen Collection


- 1** Pinche el palillo de muestreo en la muestra fecal unas 5 ó 6 veces en diferentes sitios.

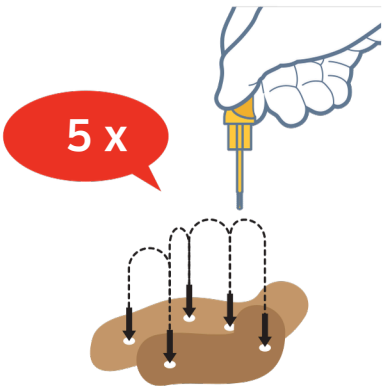

- 2** lleve la varilla de muestreo al tubo con el bufer de extraccion

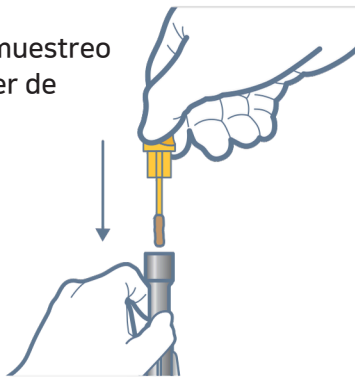

- 3** cierre la rosca

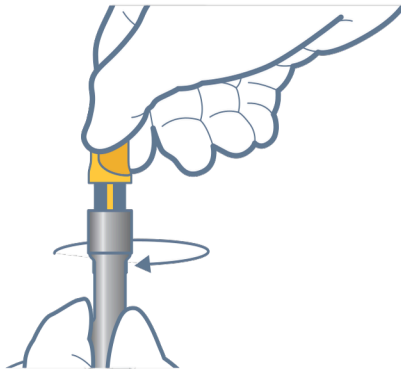


II. Test Procedure

- 1** agite el tubo de tampón de extracción ensamblado unas 10 a 15 veces



- 2** Rompa la punta negra en el exterior de la tapa negra.

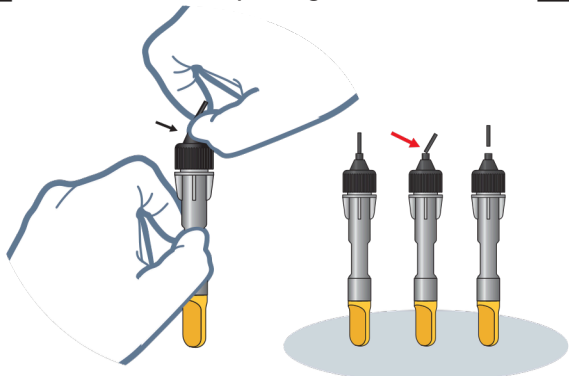

- 3** descarte 3 gotas



- 4** aplique 4 gotas de la mezcla en el tubo detector que contiene el gránulo.



- 5** mezclar 20 veces




- 6** transferir 75 μ L



- 7** coloquelos en la placa




- 8** 12 min a temperatura ambiente

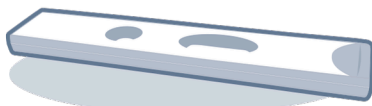


12 Min

- 9** inserte en el equipo

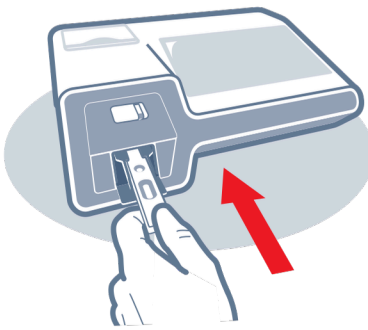

- 10** seleccione iniciar

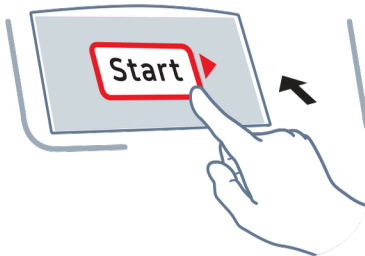

- 11** lea el resultado

