


RF IgM

Test Components

This is not a complete instruction for use. For more detailed instructions, please refer to IFU.

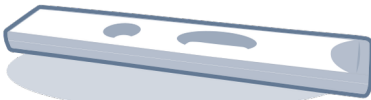

Test cartridge



Detector tube (Granule)



Detector diluent



ID chip




Ensure that the lot number of the components match.

LOT Matches **LOT**



If extraction buffer tube are stored in refrigerator, keep the sample collection tube at room temperature for 30 minutes just prior to the test.

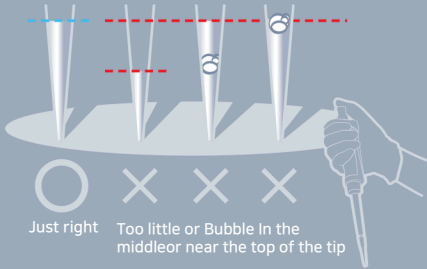

Test Procedure


1 Extraer 150 (Diluyente detector)




2 Añadir al tubo detector y disolver completamente. ** Úselo inmediatamente dentro de los 30 segundos.



3 tomar

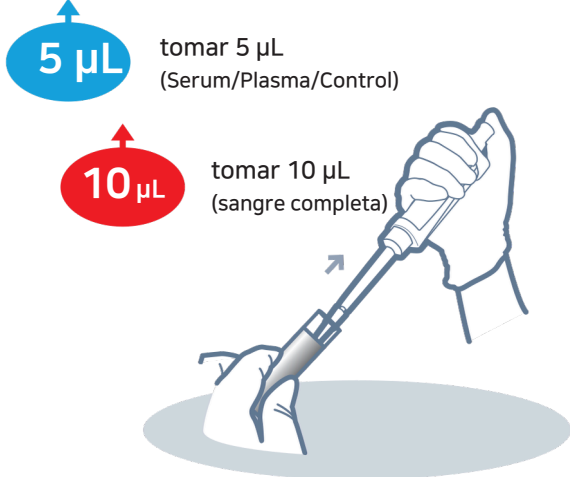


4 Agréguelo a

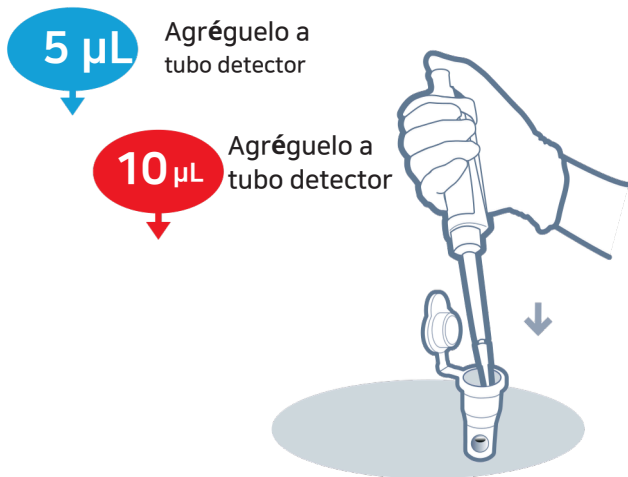

5 Cierra la tapa y agita 10 veces. ** Úselo inmediatamente dentro de los 30 segundos.



6 Extraer la mezcla

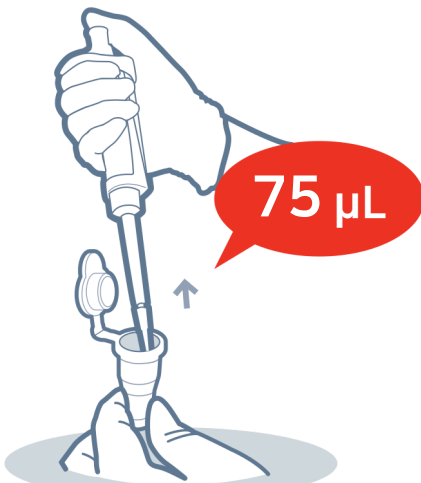

7 Cargue la mezcla de muestra




Multi modo


8 espere 5 minutos



9 inserte en el equipo

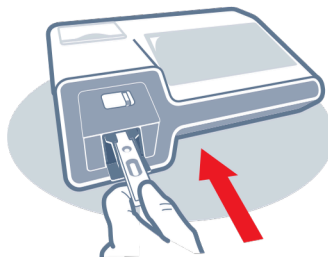

10 presione iniciar

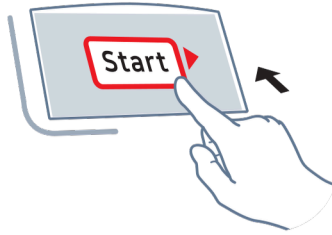


11 lea el resultado




modo unico

8 inserte en el equipo

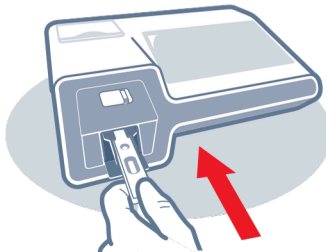

9 presione iniciar

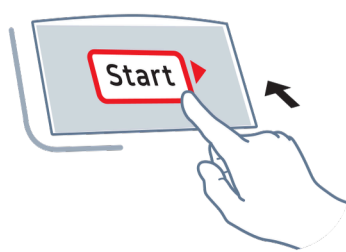

10 espere 5 minutos



11 lea el resultado

