

ichromα™

Testosterone


This is not a complete Instruction for use. For more detailed instructions, please refer to IFU.

Caution
Before the test, set
i-chamber at 25°C



25°C

Test Components

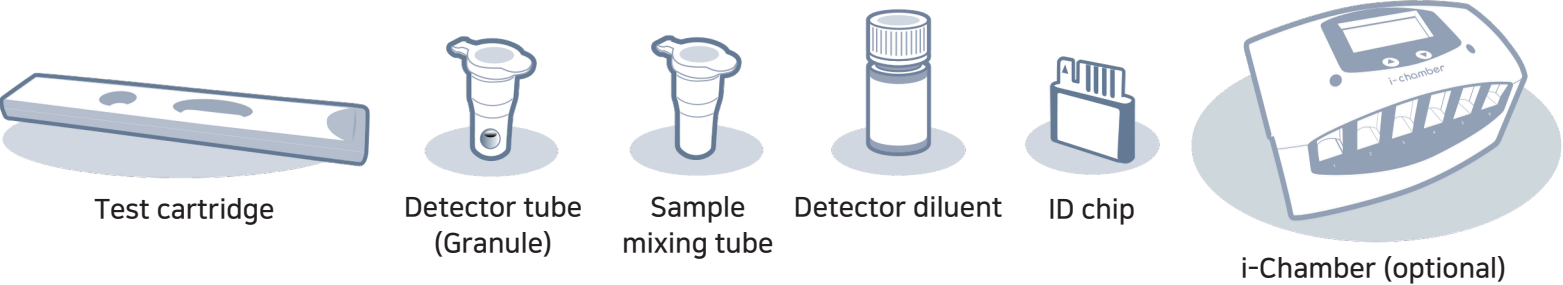


Ensure that the lot number of the components match.

LOT Matches **LOT**



If extraction buffer tube are stored in refrigerator, keep the sample collection tube at room temperature for 30 minutes just prior to the test.

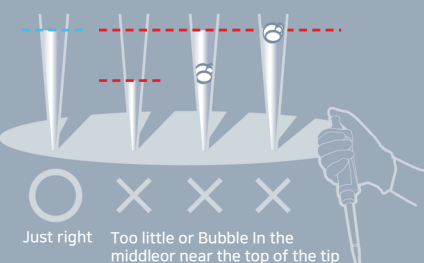

Test Procedure

*Immediate action required in these steps for accurate result

- 1 Extraer 60 μ L (Diluyente detector)

60 μ L
- 2 Agréguelo al tubo de mezcla de muestras

60 μ L
- 3 Extraer 150 (Suero/Plasma/Control)

150 μ L
- 4 Agréguelo al tubo de mezcla de muestras.

150 μ L
- 5 mezclar 10 veces

10 times
- 6 Espere 3 minutos.

3 Min
- 7 tome 150 μ L.

150 μ L
- 8 Agréguelo al tubo detector(gránulo)

150 μ L
- 9 mezclar 10 veces.

10 times
- 10 Cargue la mezcla en la placa

75 μ L
- 11 incubar 12 minutos

12 Min
- 12 inserte al equipo
- 13 seleccione iniciar
- 14 lea el resultado